

LIMITED WARRANTY

This blood pressure instrument is warranted against defects in materials and workmanship under normal use and service as follows:

1. Warranty service extends to the original retail purchaser only and commences with the date of delivery.
2. The optical system is warranted for one year.

What is covered: Repair, or replacement of parts, and labor.

What is not covered: Transportation charges. Damage caused by abuse, misuse, accident, or negligence. Incidental, special, or consequential damages. Some states do not allow the exclusion or limitation of incidental, special, or consequential damages, so this limitation may not apply to you.

To Obtain Warranty: Send item(s) postage paid to ADC. Attn. Repair Department, 55 Commerce Drive, Hauppauge, NY 11788. Please include your name and address, phone no., proof of purchase, and \$2.00 to cover the cost of return shipping and handling.

Implied Warranty: Any implied warranty shall be limited in duration to the terms of this warranty and in no case beyond the original selling price (except where prohibited by law). This warranty gives you specific legal rights and you may have other rights which vary from state to state.

**Warranty Service Center
55 Commerce Drive
Hauppauge, NY 11788**

Printed in USA Part#9362A-00

©Copyright ADC2002

Pocket Otoscope

Your Otoscope (Model #5211) consists of a "C" battery handle. Otoscope Instrument Head with removable 4x viewing lens, reusable specula (2.5mm, 3.5 and 4.5mm), and insufflator fitting, and leatherette carrying case. (Veterinary Model equipped with a 4mm, 5mm and 7mm speculum).

Installing Batteries: Remove the knurled cap at the base of the handle by turning counter-clockwise. Insert two fresh "C" cell batteries (not included) positive polarity towards the instrument head. Replace battery compartment cap by turning clockwise.

Turning instrument on: To illuminate, press the white power switch and rotate the rheostatic bezel clockwise until desired lamp intensity obtained (0 position is off, red 1 position is full power). Slide the switch in the opposite direction to turn off.

Attaching the specula: Push the flanged end of the desired specula onto the instrument head aligning the slot in the specula flange with the rivet at the top of the instrument head and twist counter-clockwise to engage. Reverse procedure to remove.

Removing Otoscope Instrument Head: The head features a bayonette style mount. To remove, push down firmly and rotate counter-clockwise. To engage, align slots on base of instrument head with tabs on bayonette. Push head and twist clockwise until engaged.

Removing Viewing lens: To remove during an examination, rotate lens counter-clockwise and pull. Reverse procedure to attach.

Replacing lamp: remove specula. Pull black sleeve off of lamp. Twist lamp counter-clockwise to remove. Replace with ADC part #5211-4. Reverse procedure to install new lamp.