

# Stryker (Gaymar) Pump to Adroit Heating Pads

If a Patient has a Stryker (Gaymar) pump but needs a new Adroit Soft-Temp heating pad they will need to either:



TP500 Pump



TP700 Pump



1. Swap out the Clik-Tite connectors on the end of the heat pump hose with Adroit STA2.1 colder connectors.

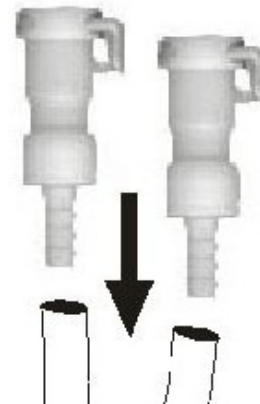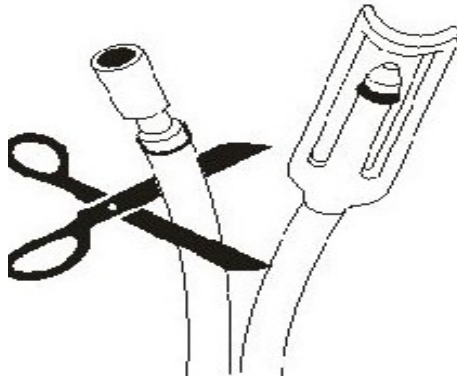

2. Use a FEMALE pig-tail adapter. The female ends connect to the Soft-Temp pad and the male/female ends connect to the pump.



Male / Female Ends to Pump

Female Ends to Pad