



CLEANING & STERILIZING INSTRUCTION:

- 1.) POLYCARBONATE COLLECTION BOTTLES CAN BE CLEANED WITH ANY MILD NON-ALKALINE DETERGENT. DO NOT USE ABRASIVES SUCH AS SCOURING POWDERS, STEEL WOOL, ALKALINE DETERGENTS OR CLEANING AGENTS NOT COMPATIBLE WITH POLYCARBONATE MATERIAL.**
- 2.) BEFORE AUTOCLAVING, BE SURE TO RINSE ANY RESIDUAL CHEMICALS OR DETERGENTS THOROUGHLY FROM POLYCARBONATE COLLECTION BOTTLES AND DRY THOROUGHLY.**
- 3.) BEFORE AUTOCLAVING, REMOVE ANY CLOSURES ON POLYCARBONATE BOTTLES, ALWAYS USE MINIMUM TEMPERATURES AND CYCLE TIMES. DO NOT EXCEED 270°.**

**ALLIED HEALTHCARE PRODUCTS
1720 SUBLETTE AVE.
ST. LOUIS, MO. 63110
TEL. (314) 771-2400**

**S188-333-001
Rev. A**