**Bovie**

**AARON 800™-EU/900™**

**HIGH FREQUENCY DESICCATOR**

**AARON 800™ - EU**

**AARON 900™**

## **USER'S GUIDE**

---

# TABLE OF CONTENTS

Introduction.....3

Precautions.....3

Unit Operation .....4

Setup Procedures .....5

Maintenance .....6

Sterilization .....6

Accessories .....7

Technical Description.....7

Warranty and Repair .....8

Troubleshooting.....8

Transport and Storage .....8

Graphs.....9

Symbol Key.....10

---

## INTRODUCTION

Thank you for purchasing the Aaron High Frequency Desiccator. Please visually check the unit to ensure that damage did not occur during shipment and that all standard items are included. The basic unit should include the electrosurgical unit with power cord, a handpiece, 50 disposable sharp dermal tips (A900 only), 50 disposable blunt dermal tips (A900 only), and a mounting kit (A837). If there are any discrepancies, please contact Aaron at +1-727-384-2323.

EU Authorized Representative:  
Peter J. Smith Medical Products Marketing  
18 Yeates Close  
Thame OX9 3AR, UK

## PRECAUTIONS

In order to safely operate the Aaron High Frequency Desiccator, several precautions need to be followed.

.....  
**CAUTIONS:**

Federal (USA) law restricts sale of this device to or on the order of a physician.

Use caution when using the Aaron High Frequency Desiccator on patients who have a pacemaker or metal implant.

The function of other electromagnetic equipment can be interfered with by the use of high frequency current.

When high frequency surgical equipment and physiological monitoring equipment are used simultaneously on the same patient, any monitoring electrodes should be placed as far as possible from the surgical electrodes. Needle monitoring electrodes are not recommended.

To avoid the possibility of an electrosurgical burn to either the patient or the physician, do not allow the patient to come in contact with a grounded metal object during activation. When activating the unit, there should be no direct skin contact between the patient and the doctor. It is also advisable to remove any loose fitting jewelry from the patient prior to activation.

Cables should be positioned in such a way that contact with the patient or other leads is avoided. Temporarily unused active electrodes should be stored isolated from the patient.

This is a non-sterile product.

System is not Mains Voltage Regulated. Recalibration is required if input Mains Voltage is changed.

To reduce the risk of electrical shock, do not remove back plate. Refer servicing to qualified personnel.

.....  
**WARNINGS:**

Do not use the Aaron High Frequency Desiccator in the presence of flammable anesthetics. Fire could result.

Keep the power setting as low as possible to achieve the desired effect.

For surgical procedures where the HF current could flow through parts of the body having a relatively small cross-sectional area, the use of bipolar techniques may be desirable to avoid unwanted coagulation.

Apparent low output or failure of the surgical equipment to function correctly at the normal operating settings may indicate faulty application of the neutral electrode or poor contact in its connections.

Store the unit in a cool, dry place.

Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".

## UNIT OPERATION

The Aaron High Frequency Desiccator produces radio frequency current which is useful for the removal and destruction of superficial cutaneous and mucosal lesions. This is done by performing desiccation and fulguration procedures. Electrosurgical desiccation occurs when the electrode is placed directly onto the surface of the lesion. Fulguration occurs when the electrode is held slightly above the lesion and an arc is delivered to the lesion. The unit also provides fast and efficient bleeding control by coagulation of capillaries and small blood vessels.

For the majority of desiccation, fulguration, and coagulation procedures utilizing the standard handpiece in the monopolar output, the patient plate is optional. When used, the patient plate will intensify the coagulation properties of the unit and also lessen the opportunity for an electrosurgical burn. The optional footswitch adds versatility when using the standard handpiece in the monopolar output, as the footswitch allows you to activate the unit by either the handpiece or the footswitch. Bipolar outputs are available for those physicians who prefer to utilize bipolar forceps to perform coagulation procedures. A footswitch is required when using the bipolar output and the patient plate is not used. Procedures that are performed in sensitive areas may require an anesthetic. Flammable anesthetics should not be used.

If you are unfamiliar with the operation of a low powered electrosurgery unit, it is advisable to practice on chicken or lean flank steak to visualize the effects at various output and power levels.

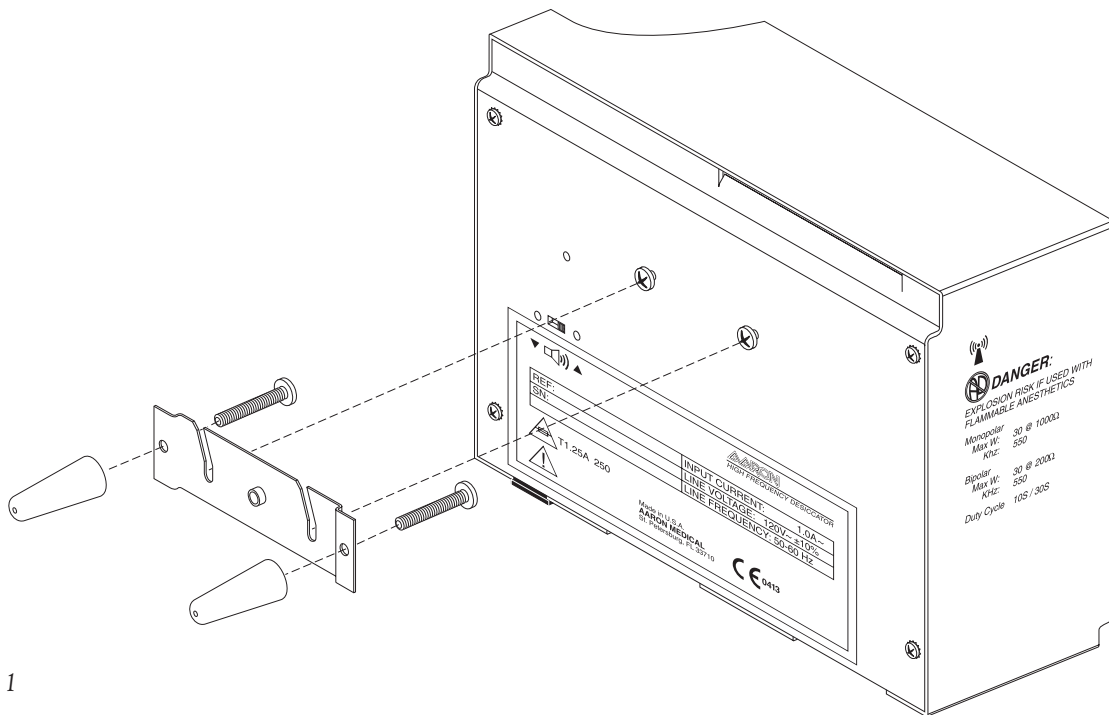
Figure 1

## SETUP PROCEDURES

1. Mount the Aaron High Frequency Desiccator on the wall or optional mobile stand using the standard mounting kit (see figure 1). Do not operate the unit in the horizontal position, as liquids may spill into unit.

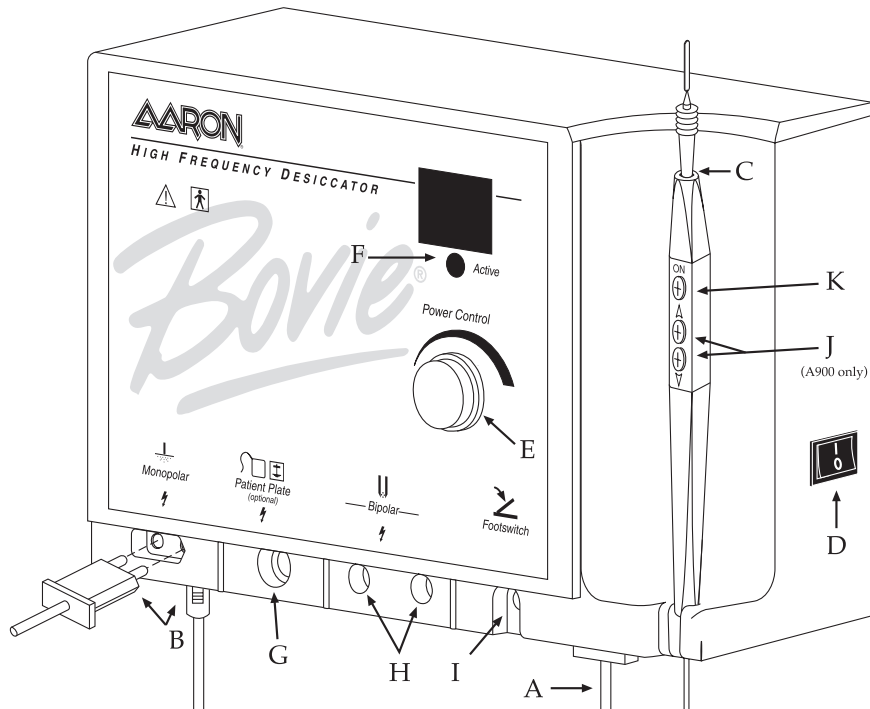
Figure 2

2. Plug the female end of the power cord into the base of the unit (see figure 2, letter A).
3. Plug the male end of the power cord into a grounded wall receptacle.
4. Plug the handpiece into the monopolar output on the lower left of the front on the unit (see figure 2, letter B). The plug is designed to fit in only one direction. Plug the smaller round connector from the handpiece (A900 only) into the receptacle on the bottom of the unit (see figure 2, letter B). The three button handpiece is designed to give the doctor complete fingertip control of the power output settings.
5. Slide the standard electrode into the handpiece until it is firmly seated (see figure 2, letter C). The handpiece will accept most standard  $\frac{3}{32}$ " electrodes.
6. Slide the handpiece from above into the holder on the right side of the unit.
7. Turn the unit power on utilizing the switch on the right side panel of the unit (see figure 2, letter D).
8. Set the power output either by using the dial on the front of the unit (see figure 2, letter E) or on Aaron 900 unit only by the up and down buttons on the handpiece (see figure 2, letter J). When power level adjustment is being made by the handpiece an audible tone will sound to indicate that the power level has been changed. Depressing and holding the up or down buttons will cause the power settings to change more rapidly for quick adjustment of the output settings. Power output is displayed in ".1" watt increments below 10 watts and in whole numbers from 10 to 30 watts.

**NOTICE:**

*The output settings can not be adjusted when the unit is being activated.*

9. To activate the unit, remove the handpiece from the holder. Place the handpiece in the desired position and depress the activation button (see figure 2, letter K). When the unit is activated, an audible tone is sounded and the blue active light will illuminate (see figure 2, letter F).
10. To use the optional grounding plate with cord (A802EU), insert the plug of the cord into the grounding plate output (see figure 2, letter G) and connect the other end into the grounding plate. The plate should be placed underneath the patient at a point where the entire plate is covered by bare skin. Do not use gels or creams on the plate.
11. To use the optional bipolar cord (A827EU), insert the plugs into the bipolar outputs (see figure 2, letter H). The cord is then plugged into the forceps. A sliding gate behind the monopolar and bipolar outputs prevents the user from using both simultaneously.
12. The optional footswitch is plugged into the footswitch output and placed on the floor (see figure 2, letter I). The footswitch can be used with monopolar procedures and must be used with bipolar procedures.
13. When the procedure is completed, turn the unit off utilizing the switch on the right side panel of the unit.
14. Return the handpiece to the holder on the right side of the unit and remove the electrode. The electrode should be disposed of after each procedure. If contamination has occurred to the handpiece, the handpiece should also be sterilized.
15. Adjustment of the audible tone is achieved by a switch located on the rear of the unit (see figure 3). Two tone choices are available, high and low. A small screwdriver will be necessary to make the adjustment.

Figure 3

## MAINTENANCE

The Aaron High Frequency Desiccator requires periodic cleaning. When the unit case requires cleaning, simply utilize a soap and water solution and wipe clean. Be careful to not have any water enter into the unit through the various openings. Dry the unit with a clean, lint-free cloth.

## STERILIZATION

The Aaron High Frequency Desiccator and standard accessories are supplied non-sterile. The handpiece may be cleaned and sterilized. Refer to the instruction sheet that accompanies the electrode and handpiece for specific instructions on cleaning and sterilization. We recommend that all contaminated electrodes and handpieces be sterilized prior to disposal.

---

## **ACCESSORIES**

The accessories listed below are original Aaron® accessories to be used with the Aaron High Frequency Desiccator. Accessories, replacement parts, and disposable items that are not listed should only be used when their safety and technical suitability have been checked. Additional accessories are available from your local Aaron dealer.

Reusable items must be checked for damage before each re-sterilization. Accidental burns can be caused by damaged accessories.

### ***Recommended, Standard Accessories***

<b>Catalog #</b>	<b>Description</b>
A801EU	Replaceable Handpiece (for use with A800EU only)
A901	Replaceable 3-button Handpiece (for use with A900 only)
A802EU	Grounding Plate with Cord
A827EU	Reusable Bipolar Cord
A804	Disposable Dermal Tips (Sharp)
A806	Disposable Dermal Tips (Blunt)

## **TECHNICAL DESCRIPTION**

### ***Mains Connection***

Main Voltage:	110 / 220 VAC
Main Frequency:	50 - 60 Hertz
Main Current:	1.00A / 800 mA Max.
Power Consumption:	80 W
Duty Cycle:	10sec on / 30sec off
Main Fuses	2 x 1.25A (slow blow) 110 model 2 x 800ma (slow blow) 220 model

### ***HF Output Characteristics***

#### **Monopolar**

Output Power:	30 W / 1000Ω
Output Frequency:	550 kHz
Peak Open Circuit Voltage:	5000 V

Refer to figures 5 and 7 for diagrams showing Monopolar output characteristics.

#### **Bipolar**

Output Power	30 W / 200Ω
Output Frequency:	550 kHz
Peak Open Circuit Voltage:	2100 V

Refer to figures 4 and 6 for diagrams showing Bipolar output characteristics.

---

## **Safety**

Basic Construction:	In accordance with EN 60601-1
Mode of Operation:	Intermittent operation
Protection Class:	CLASS I EQUIPMENT
Output Type:	TYPE BF

## **Dimensions and Weight**

Length x Width x Height = 9" x 4.5" x 6.3"

Weight: 5 lbs.

## **WARRANTY AND REPAIR**

The Aaron High Frequency Desiccator is covered under warranty for a period of two years. The handpiece is covered under warranty for a period of one year or 25 steam autoclave cycles, whichever comes first. The warranty becomes null and void if damage occurs from incorrect handling or misuse of the product.

For warranty and repair work, please contact Aaron and obtain a Return Goods Authorization number (RGA). Place the number so that it can be seen on the exterior of the package and ship directly to Aaron. A return without an RGA may not be accepted.

## **TROUBLESHOOTING**

The Aaron High Frequency Desiccator has been designed and manufactured with the utmost safety in mind. The unit is equipped to automatically detect a malfunction. The following table list error codes, their meaning and recommended actions to be taken to resolve the error.

<b>Error Code</b>	<b>Description of Error</b>	<b>Recommended Action</b>
E1	Activation upon power up	<ul style="list-style-type: none"><li>• Check handpiece for activation.</li><li>• Check footswitch for activation; once the activation is halted the unit will resolve the error. If the error persists the handpiece could be malfunctioning and may need to be replaced.</li></ul>
E2	DC Supply Over Voltage Detection	<ul style="list-style-type: none"><li>• Switch unit off and on again.</li><li>• Make sure unit is connected to correct power source for the unit.</li></ul>
E3	Pulse Width	<ul style="list-style-type: none"><li>• Switch unit off and on again.</li></ul>
E4	Delta Error	<ul style="list-style-type: none"><li>• Switch unit off and on again.</li></ul>
E5	Multiple Errors	<ul style="list-style-type: none"><li>• Switch unit off and on again.</li></ul>
E6	Temperature Error	<ul style="list-style-type: none"><li>• Switch unit off. Allow unit to cool. Switch unit on.</li></ul>

If problems persist, the unit should be taken out of service and the manufacturer should be notified. For technical support or return authorization phone +1-800-537-2790.

## **TRANSPORT AND STORAGE**

Ambient temperature range: -40°C to 70°C

Relative humidity: 10% to 100%, non-condensing

Atmospheric pressure: 500hPa to 1060hPa.

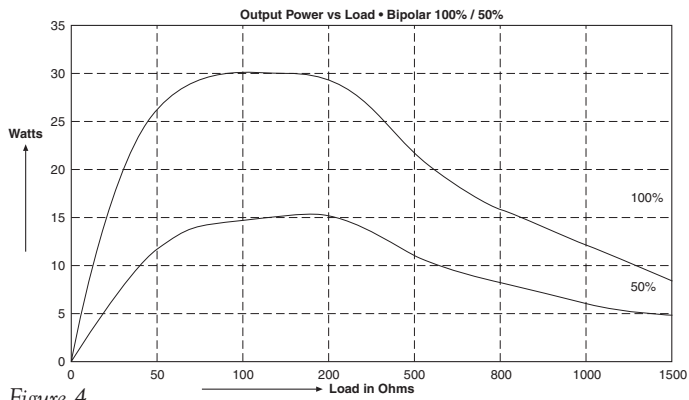
Figure 4

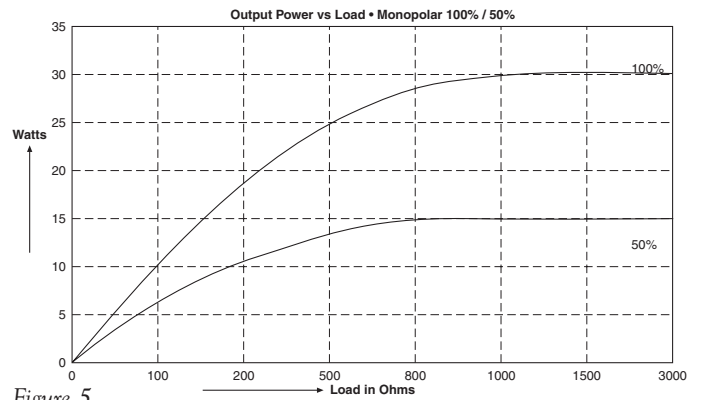
Figure 5

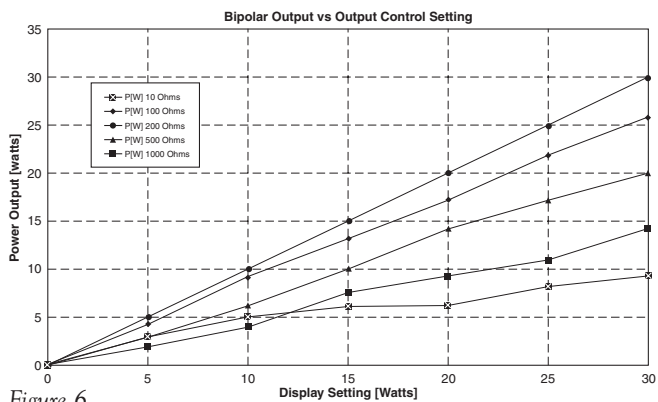
Figure 6

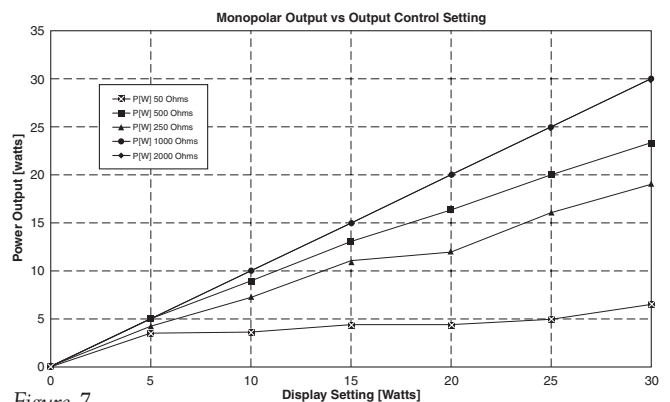
Figure 7

---

## DESCRIPTION OF SYMBOLS

Caution: To reduce the risk of electric shock, do not remove rear panel.  
Refer all servicing to qualified personnel.

Caution: Jack location is a source of dangerous voltage.

Caution: Read directions for use prior to using equipment.

On (power: connection to the mains).

Off (power: disconnection from the mains).

Fuse enclosed (warning appears next to symbol).

Monopolar output jack (hand control pencil jack).

Bipolar output jack.

Patient Plate, for use with Monopolar modes.

Footswitch jack, for foot controlled activation of monopolar (optional) and bipolar devices.

Type BF Equipment.

Non-ionizing radiation.

Neutral Electrode referenced to earth.

▼ Volume control.

Explosion Risk if used with flammable anesthetics.

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