

## Clinton Industries General Care and Cleaning Guide

Step 1: For light soiling, a solution of 10% household liquid dishsoap in warm water, applied with a soft damp cloth. Rinse with clean water and dry.

Step 2: For heavy soiling, dampen a soft white cloth with a one to one (1:1) solution of Fantastik® and water or Formula 409® and water. Rub gently and rinse with a water dampened cloth.

Step 3: For more difficult stains, dampen a soft white cloth with a solution of household bleach (10% bleach / 90% water). Rub gently and rinse with a water dampened cloth to remove bleach concentration.

Do not use with alcohol based cleaning agents!

©1997 Spradling International, Inc.®

This information is not a guarantee and does not relieve the user from the responsibility of the proper and safe use of the product and all cleaning agents. The use of certain agents can be harmful to the surface appearance and lifespan of vinyl. Spradling, its agents, and assigns assume no responsibility resulting from the use of such cleaning agents to the vinyl. Please check compatibility when using this product in combination with painted or varnished surfaces. Formula 409® is a registered trademark of The Clorox Company® Fantastik® is a registered trademark of DowBrands, Inc.®, Registered trademark of the Dow Chemical Co.® Spradling International is not sponsored by, affiliated with, or endorsed by such trademark owners. Additionally, any use by Spradling International of any third-party trademark does not constitute any endorsement by Spradling International of such third party.